Absolute Kontraindikationen für den externen Sinuslift Absolute contraindications for external sinus lifts
 Contraindicaciones absolutas para la elevación externa del seno:

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- Insufficient supply of bone (a residual bone height of at least 3 mm is required for immediate transplantation)
- Not recommended in the case of complex morphology of the sinus floor (septum)
- Diseases that impair bone metabolism
- Dentoalveolar growth which has not come to an end (exception: Cases in which no dentoalveolar growth can be expected, e.g. ectodermal dysplasia)
- Active infections as well as pathological processes

NOTE: A simple mucocele in the preparation area (sinus floor) does not represent any contraindication and can frequently be removed by means of aspiration.

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- Residual bone height under 5 mm
- Not recommended in the case of complex morphology of the sinus floor (septum)
- Dentoalveolar growth which has not come to an end (exception: Cases in which no dentoalveolar growth can be expected, e.g. ectodermal dysplasia)
- Active infections as well as pathological processes

NOTE: A simple mucocele in the preparation area (sinus floor) does not represent any contraindication and can frequently be removed by means of aspiration.

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CAUTION: Lifting the Schneiderian membrane must take place very carefully with a low level of pressure. A perforation should be avoided.

NOTE: Related contraindications and further instructions deemed set-specific and generally valid can be found at the download area of our website at "Application and Safety Instructions for Bone Management Systems™".

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¹ 204=RA, 205=RA L, 106=HP XL

² Largest working part diameter in 1/10 mm

Fig.	SL25	SL30	SL35	SL40	SL45	SL50	SL0
Size ²	025	030	035	040	045	050	
Length mm	16.0	16.0	16.0	16.0	16.0	16.0	

Fig.	TL1	TL2	TL3	TL8	TL19	TL20
Name	Sinuslift 1 Sinuslift 1	Sinuslift 2 Sinuslift 2	Sinuslift 3 Sinuslift 3	Condenser Verdichter	Applicator Applikator	Luniatschek Luniatschek
Length mm	173.0	173.0	172.0	181.0	182.0	160.0

Optional erhältlich: Hammer Optional available: Hammer Opción disponible: Martillo

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- Optimally matched instruments for the easy execution of internal sinus floor elevation
- Optimally matched instruments for the easy execution of external sinus floor elevation are also included
- Ideal condensation of the implant site at the internal sinus lift procedure
- Easy harvesting and generation of bone graft material

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Combined Sinuslift System

85FLBM07 - 0315

Bone Management® is a registered trademark of the Hager & Meisinger GmbH, Germany

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The Bone Management® Lift-Control Plus system offers instruments for the Summers technique of internal / indirect sinus lift¹ and for Boyne direct sinus lift². Lift-Control Plus is a combination of the approved Bone Management® systems, External-Lift-Control and Internal-Lift-Control.

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¹ Summers RB. A new concept in maxillary implant surgery: the osteotome technique. Compendium 1994 Feb;15(2):152, 4-6, 8 passim; quiz 62.

² Boyne PJ, James RA. Grafting of the maxillary sinus floor with autogenous marrow and bone. J Oral Surg 1980 Aug;38(8):613-6.

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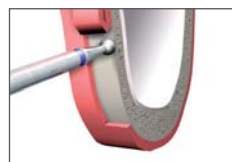
The instruments used are indicated for performing an external sinus lift in the case of insufficient bone height in the lateral maxillary region of the teeth. After creating a lateral window, the Schneiderian membrane is carefully removed using various hand instruments and carefully lifted into the sinus cavity. Additionally, the CARBOCER®-coated instruments provide a good level of contrast for reflection-free work to be performed. The space created can be subsequently filled with augmentation material.

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carefully and the bone thinned down to the membrane in this region. Attention must be paid to flapping in the window in an upward direction.

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The Schneiderian membrane is removed using CARBOCER®-coated instruments. Depending on requirements, hand instruments are available in various sizes and angles in order to loosen the mucosa of the maxillary sinus carefully and without injury and lift it in an upward direction.

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
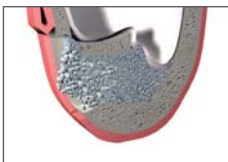
The previously opened bone window in the maxillary sinus wall can now be transferred upwards carefully in the maxillary sinus together with already separated mucosal membrane.

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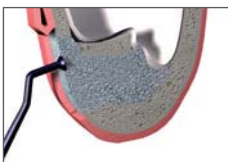The created cavity in the maxillary sinus can now be loosely filled with appropriate graft material. This should support the previously lifted bone window. The inserted graft can now be condensed carefully using the Condenser TL8.

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In case of direct implantation, the implant is placed in the newly created bone site and the remaining open bone access is covered with a membrane, if required. Then the mucoperiosteal flap is sutured saliva-tight.

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After an appropriate healing period, the implant(s) is/are uncovered followed by appropriate prosthetic restoration. In the two-stage process, implantation is performed only after complete ossification of the graft.

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The instruments used in the described application are indicated for carrying out internal sinus lifts where the bone in the implant bed is driven upwardly into the direction of the maxillary sinus. Summers developed this technique allowing transcrestal access to the maxillary sinus. Augmentations of smaller vertical defects (3 mm-4 mm) in the sinus at a residual bone height of more than 6 mm are deemed an indication for this procedure with an implantation at the same time depending on the case situation.

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The use of a pre-punch creates a guide channel to prevent any slipping of the twist drill.

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Pre-drilling is then performed with a suitable pilot drill with depth markings. The depth and diameter depend on the available vertical and horizontal bone dimensions, whereby the drill hole should correspond to up to two thirds of vertical bone height (check with x-ray). In as far as no simultaneous horizontal condensation is required, use the expanders to widen the drill hole to the implant diameter according to the manufacturer's protocol.

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A further successive preparation of the implant site shall take place with increasingly larger elevators. Treatment occurs up to about 1 mm beneath the sinus floor. The sinus floor including the apically condensed bone is driven cranially into the sinus with the aid of the previously used elevator. The Schneiderian membrane is thus raised away from the maxillary sinus floor.

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During the single-stage procedure, the newly established space is filled with a graft and an immediate insertion of the implant is performed.

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During the double-stage procedure, as an initial step, the newly established room is filled with a graft. The implantation takes place in a second intervention as soon as the new formation of bone has concluded.

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