Tissue-Control

Advanced Augmentation System

MEISINGER
GERMANY / USA

illustrated 1:4
 Handinstrumente
 Hand instruments
 Instrumentos de mano


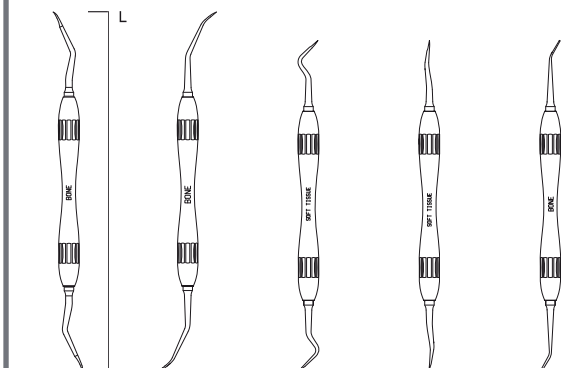
Fig.	HI020	HI021	HI022	HI023	HI024
Length mm	190,0	190,0	175,0	175,0	175,0
	Bone	Bone	Soft Tissue	Soft Tissue	Bone
	side buc / front buc	side ling / pal	side	front	Papillaer elevator

Gabelaufnahme für Spreading-Keile
 Fork adapter attachment for spreading-chocks
 Retenedor de horquilla para cuñas de expansion

Art.-No. BMR01

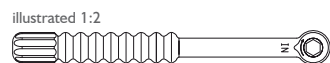Ratsche für Adapter
 Ratchet for adapter
 Carraca para adaptors

Art.-No. CARA4

Schraubendreher für Schrauben aus Screw System TX
 Screw driver for screws from Screw System TX
 Atornillador para tornillos de Screw System TX

Art.-No. TL0T1

Spreader
 Spreader
 Spreader



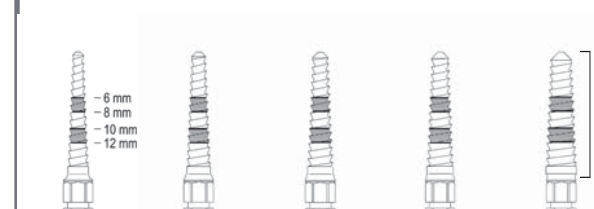
Fig.	A1005	C1005	D1005	E1005	F1005
Length mm	15,0	15,0	15,0	15,0	15,0
	2,70	3,10	3,30	3,50	4,00
	1,43	1,84	2,05	2,26	2,79

External diameter
 Minimal diameter

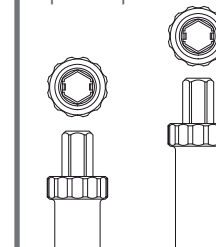Adapter für Ratsche
 Adapter for ratchet
 Adaptadores para carraca

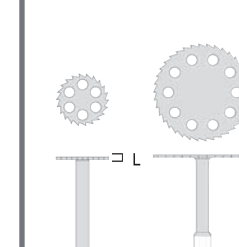Diamantierte Sägen
 Diamond coated saws
 Sierras diamantadas

Spreading-Keile (3 St.)
 Spreading-chocks (3p.)
 Cuñas de expansion (3 pzs.)

CE 0044

Fig.	CA1RB	CA0RB	231DC	231DC	BMR02
Shank ¹			204	204	
Size ²	short	long	070	130	
Length mm			0,3	0,3	11,5
					6,0

¹204=RA ²Largest working part diameter in 1/10 mm

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Please follow general application and safety instructions for MEISINGER products in the medical area and also the advice for reprocessing (cleaning, disinfection and sterilisation) of medical devices from Hager & Meisinger GmbH.

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One set for everything! This system offers applications for modern augmentation techniques for hard and soft tissue. The Bone Management® set Tissue-Control allows all modern augmentation techniques to be carried out easily and safely in bone and soft tissue. The design of the various hand instruments is ideally suited for the tunnel technique or envelope technique, both of which represent minimal invasive methods for augmenting hard and soft tissue and which minimize resorption and shrinkage of grafts. In addition, the system enables bone spreading using a modified extension graft.

The set consists of 3 sections:

1. Hand instruments for transplanting autologous bone plugs for vertical and / or horizontal augmentation employing the tunnel technique
2. Hand instruments for transplanting soft tissue with the tunnel or envelope techniques
3. Instruments for modified extension grafts

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- One set for 3 applications
- Optimal applications for modern augmentation techniques for hard and soft tissue
- Easy and safe to use
- Hand instruments for transplanting autologous bone plugs for vertical and / or horizontal augmentation employing the tunnel technique
- Hand instruments for transplanting soft tissue with the tunnel or envelope techniques
- Instruments for modified extension grafts

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Bone augmentation employing the tunnel technique

The specially arched and finished Tissue-Control hand instruments can be used for tissue-saving in every jaw region to prepare a tunnel with one or two small incisions in the mucosa to accept and mount a bone graft. With this technique, the graft is covered by the periosteum and thus protected against resorption.

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Thickening of soft tissue and recession coverage employing the tunnel or envelope techniques

The specially arched and finished Tissue-Control hand instruments can be employed in a tissue-conserving manner in every region of the jaw to prepare a tunnel or envelope with one or two small incisions to accept a connective tissue graft. Employing this technique, the soft tissue can be thickened or a recession covered at the tooth or implant. By preparing the tunnel between the periosteum and the mucosa, the connective tissue graft is positioned on the periosteum and covered by the mucosa and results in no bone resorption and only marginal shrinkage of the connective tissue.

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Vertical incision in the mucosa or incision in the vestibular periodontal pocket.

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Preparation in the attached gingiva using a micro scalpel supraperiostally.

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Further preparation of the tunnel or envelope in the mucosa in the region of the anterior teeth using hand instrument HI023.

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Further preparation of the tunnel or envelope supraperiostally in the mucosa in the lateral region of the teeth using hand instrument HI022.

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Removal of the connective tissue graft employing the same technique palatally to the extracted tooth.

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Delivering and fixing the graft with a suture.

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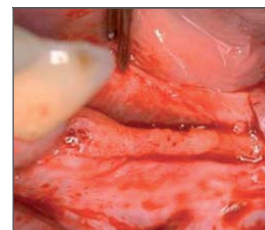
Modified extension graft to spread the bone

Employing this minimal invasive method allows careful spreading of the bone up to a remaining thickness of the jaw bone of 3 mm. As this is performed with vertical relief incisions without separating the vestibular periosteum, the defined bone area can be spread more than with conventional bone spreading techniques as the buccal bone lamella is covered by the periosteum and thus protected against resorption. In addition, the special screw-shaped condensing and spreading instruments (spreader) in this set can be used to spread horizontal resorbed bone in a controlled manner and to condense cancellous bone gently. Following controlled bone extension, any of the conventional implant systems available on the market can be delivered.

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Further preparation of a mucosal flap to the fold. The periosteum and the submucosal connective tissue remain on the bone.

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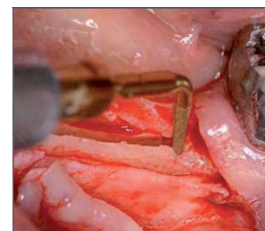
Crestal incision in the bone using a diamond saw from the set or a piezoelectrically operated unit.

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
Vertical relief incisions with an osteotomy instrument or piezoelectrical device mesially and distally in the bone limit the extension area from crestal to vestibular.

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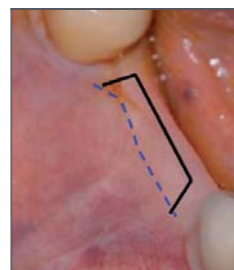
Incision on the alveolar ridge and preparation of a mucoperiosteal flap up to the mucogingival junction.

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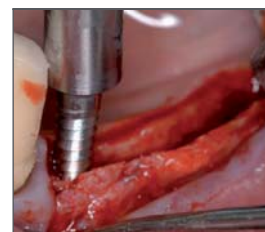
Stepwise spreading of the bone using spreaders with increasing diameters with ratchet CARA4 and the corresponding adapter CA1RB or CA0RB. For bone qualities of types 1-2, the first pilot hole should be drilled with a drill from the planned implant system prior to using the spreaders.

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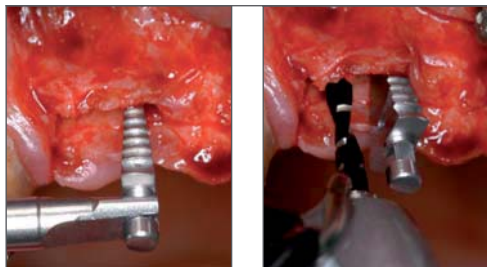Using the specially designed wedge spreader BMR02, which can be placed between the planned implant positions with the fork adapter attachment BMR01, the extended bone area is kept accessible to allow careful

preparation of the implant site with increasing drill sizes without damaging the buccal bone lamella.

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If required, the remaining bone cavities can be filled with a suitable graft. The use of membranes is possible. The special flap preparation simplifies tension-free and tight suture closure.

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From 1983 - 1991 scientific assistant in the prosthetic and surgical department of Westfälische-Wilhelms-University Münster. Since 1991 working in a group practice with Dr. Bernard Drücke and Dr. Joseph Janzen in Münster. 1991 graduated as a dentist specializing in oral surgery, since 1999 certified consultant for implants, authorization for further training in „oral surgery“ by the Dental Association of Westfalen-Lippe. Consultant of the Curriculum for Implantology in the BDIZ at the University of Cologne and in F.I.R. Institute Recklinghausen. Lecturing at the Staff College of Dentists Association of Westfalen-Lippe, and at numerous national and international conferences. The BDO appointed expert in implantology. Together with Prof. Topoll implementation of Münsteraner Symposium on Implantology and Periodontology. Numerous practical courses in the subject area of implant dentistry and oral surgery. Member of ECDI (European Center of Dental Implantology).

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General instructions:

- All products delivered are unsterile, therefore, before initial and each further potential application, products should be treated (cleaning / disinfection / sterilization).

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Art.-No.: BT100

Bone Management® is a registered trademark of Hager & Meisinger GmbH, Germany

85FL085-0212

Hager & Meisinger GmbH

Hansemannstr. 10
41468 Neuss • Germany
Tel.: +49 (0) 21 31-20 12 0
Fax: +49 (0) 21 31-20 12 222
Internet: www.meisinger.de
E-mail: info@meisinger.de

Meisinger USA, L.L.C.

7442 South Tucson Way, Suite 130
Centennial, Colorado 80112 • USA
Tel.: +1 (303) 268-5400
Toll free: +1 (866) 634-7464
Fax: +1 (303) 268-5407
Internet: www.meisingerusa.com
E-mail: info@meisingerusa.com