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DE / EN

## Turbine

---

L400M  
L400S

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SycoTec GmbH & Co. KG

Wangener Str. 78

88299 Leutkirch

DEUTSCHLAND

Telefon: +49-7561-86-0

Fax: +49-7561-86-400

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Telefon: +49-7561-86-0

Fax: +49-7561-86-400

Condura is a trademark of Hager & Meisinger GmbH.  
All other trademarks referred to in this handbook are the  
property of their respective owner.

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Copyright 2013 MEISINGER. All rights reserved.

## Table of Contents

1	Important Information.....	35
	Symbols used.....	35
	Safety measures .....	38
	Proper use – intended use.....	39
	Additional information for use and safety .....	41
	Target group .....	41
2	Technical data .....	42
3	Processing methods .....	44
	Preparation at the location of use .....	44
	Cleaning.....	44
	Sterilisation pursuant to ISO 17665 .....	46
4	Installation and commissioning .....	49
	Supply of coolant .....	50
	Use of diamond and hard metal drill .....	51
5	Operation .....	52
	Connection to units .....	52
	Attaching the turbine .....	52

## Table of Contents

Pulling off the turbine .....	54
Attaching the drill.....	54
Removing the drill .....	55
6 Maintenance and repair .....	56
Care products and maintenance systems.....	56
Care with maintenance systems: <i>STATMATIC</i> / <i>STATMATIC PLUS</i> .....	57
Care with dental care spray .....	58
7 Guarantee .....	59

# 1 Important Information

Dear user,

we hope that you enjoy using your new quality product. In order to ensure that you use the product correctly, efficiently and safely, please adhere to the following information.

## Symbols used

In the manual		On the packaging	
	Situation which can result in injury or danger to persons, when the instructions are not followed.		Keep dry
	Important information for users and technicians		Maximum stacking load
	Instructions for disposal		Temperature range
	Steam-sterilisable up to 135 °C (275 °F)		Humidity

## 1 Important Information

In the manual		On the packaging	
	CE marking		Quantity
	Thermodisinfected		
	Manufacturer		
	Follow instructions for use!		

Product subject to technical modifications and design deviations.

No retrofit warranty on existing devices.

The manufacturer does not accept any responsibility for damages caused by:

- external force (poor quality of media or faulty installation)
- use of wrong information

## 1 Important Information

- unintended use
- improper repairs

Repair and maintenance, except for the activities described in these instructions for use, must only be performed by qualified professionals.

- Changes made by third parties render the approval null and void.
- Use original parts and spare parts only.

The guarantee becomes ineffective if defects or their consequences are caused by manipulation or modification of the product by the customer or third parties not authorised by us.

Disposal of devices and accessories after use:

Based on EC directive 2002/96/EC on waste electrical and electronic equipment we hereby inform you that this product is subject to the aforementioned directive and must be disposed of through special channels. Contact the manufacturer or dental supply shop for further information.

# 1 Important Information

## Safety measures

 **WARNING! Risk by damage, irregular operating noise, excessive vibrations or unusual heating!**

Risk for treating personnel and patients.

▷ Immediately stop work and contact maintenance.

▷ Before longer periods of non-use the turbine must be cleaned, maintained and kept in a dry place in accordance with instructions.

**NOTE! Risk by improperly stored turbine!**

Damage to turbine caused by fall.

▷ Turbine must be properly placed in the tray after treatment.

# 1 Important Information

## Proper use – intended use

This turbine is:

- intended only for dental treatment within the scope of dentistry. Any type of misuse or modifications of the product are not permitted and may cause danger.
- a turbine in accordance with applicable legal national regulations.
- a dental handpiece, air driven with lighting, clamping mechanism with push button, 1-4 internal spray nozzles.
- not allowed for use in explosive environments.
- to be used by observing the manufacturer's instructions for use.

The turbine is intended for the following applications:

- **Indications:** Intended for the removal of carious material and solid dental substances, for the preparation of cavities, to finish dental fissures and polish dental surfaces.
- **Contraindications:** currently none known.
- **Adverse reactions:** currently none known.

## 1 Important Information

According to these regulations this medical product must only be used for the described applications by professional technicians and under consideration of the following issues:

- applicable industrial safety regulations,
- applicable accident prevention measures and
- these instructions for use.

According to these regulations it is the user's responsibility:

- to use only faultless work equipment,
- to ensure the correct purpose of use,
- to protect the patient and others from danger and
- to avoid contamination by the product.

# 1 Important Information

## Additional information for use and safety

When the turbine is not in use over a longer period of time:

- ▶ Clean and maintain the turbine in accordance with the instructions for use and store in a dry place.

In case of damage, heating, irregular operating noise, excessive vibrations, too large drilling impact or insufficient fixation of the drill:

- ▶ Do not use the device.
- ▶ Contact customer service or authorised dealer to avoid further damage.

## Target group

This document is directed to the dentist and the dentist's assistant.

The chapter "Commissioning" is additionally directed to the service technician.

## 2 Technical data

Operating pressure	2.1 – 4.2 bar (30 – 60 psi)
Recommendation	> 2.8 bar (> 40 psi)
Air consumption	39 – 55 SLPM
Idle running speed (L400M)	330,000 – 410,000 min <sup>-1</sup>
Idle running speed (L400S)	380,000 – 460,000 min <sup>-1</sup>
Recommended working pressure	2 – 3 N
Weight	77 g (0.169 lb)
Dimensions	120 x 40 x 16 mm (4.72 x 1.56 x 0.63 inch)
Noise level	< 70 dB

## 2 Technical data

### Environmental conditions

---

Installation site:	Permissible in interior spaces
Ambient temperature:	10 – 40 °C (50 – 104 °F)
Relative air humidity:	30 – 75 %

### Storage and transport conditions

---

Ambient temperature:	-30 – 70 °C (-22 – 158 °F)
Relative air humidity:	15 – 93 %
Compressed air:	700 – 1,060 hPa
Keep dry!	

Technical modifications reserved.

### 3 Processing methods

In the following the processing methods according to ISO 17664 are described.

#### Preparation at the location of use

**WARNING! Danger from non-sterile products!**

Contaminated medical products pose a threat of infection.

- ▷ Take suitable action to protect personnel.
- ▷ Immediately remove residues of cement, composite or blood at the place of use.
- ▷ Transport the medical product in dry condition for processing.
- ▷ Do not soak in solutions.
- ▷ The medical product must be processed right after the treatment.

#### Cleaning

**NOTE! Malfunctions by cleaning in ultrasound device!**

Defects of the medical product.

- ▷ The medical product must not be placed in a cleaning basin or ultrasound device.


### 3 Processing methods

#### Option I

#### Manual exterior cleaning

Necessary accessories:

- Potable water  $30\text{ }^{\circ}\text{C} \pm 5\text{ }^{\circ}\text{C}$  ( $86\text{ }^{\circ}\text{F} \pm 9\text{ }^{\circ}\text{F}$ )
- Brush, e.g., medium-hard toothbrush

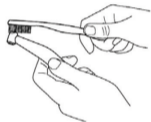- ▶ Brush under running potable water for 20 seconds.

#### Manual interior cleaning and maintenance (for U.S. market)

- ▶ Use STATCARE spray in accordance with the instructions for use.

#### Option II

#### Automated internal and external cleaning and disinfection in a autoclave pursuant to ISO 15883

#### **NOTE! Wet medical product after processing!**

Defects with the use of wet medical product.

- ▷ Choose suitable cleaning and drying method.
- ▷ Ensure that the medical product is completely dry after processing.

- In some countries the applicable national guidelines require automated processing in a validated cleaning and disinfection device (CDD) such as, for instance, SciCan HCDRIM M2 or C61.
  - ▷ Use only CDDs with integrated drier.
  - ▷ Select an appropriate program on the CDD with drying cycle.
  - ▷ Observe national guidelines.

### 3 Processing methods

Option I

#### Automated interior cleaning and maintenance

- ▶ Use an automated cleaning and maintenance device, e.g., SciCan *STATMATIC PLUS*, in accordance with the instructions for use.

#### Disinfection

The medical product can be disinfected in a steam steriliser, e.g., SciCan *STATIM*.

- ▶ Observe national and regional guidelines.

#### Sterilisation pursuant to ISO 17665

#### Packaging

The packaging of sterile goods must be adapted to the instrument in order to have a tension-free packaging.

The packaging of sterile goods must fulfil the applicable standards regarding quality and use and must be suitable for the sterilisation method!

Option II

#### Automated maintenance

- ▶ Use an automated maintenance device, e.g., SciCan *STATMATIC*, in accordance with the instructions for use.

### 3 Processing methods

- ▶ Package the medical products individually in packaging for sterile goods.
- ▶ Perform sterilisation in a steam steriliser (autoclave) pursuant to EN 13060 following ISO 17665-1.

**NOTE! Premature wear and malfunctions by improper maintenance and care!**

Reduced life span of the product.

- ▷ Treat medical product with dental cleaning products before each sterilisation cycle.

**NOTE! Contact corrosion caused by moisture!**

Damage to the medical product.

- ▷ Immediately remove the medical product from the steam steriliser after the sterilisation cycle.

The medical product can be sterilised at a maximum temperature of up to 138 °C (280.4 °F).

The medical product can be sterilised in steam sterilisers.

- ▶ Observe national and regional guidelines.
- ▶ Observe the instructions of the autoclave manufacturer.

Sterilisation in a standards-compliant (e.g. EN 13060) steam steriliser with validated B or S cycle.

### 3 Processing methods

Recommended sterilisation methods or parameters for unpackaged and packaged loads are as follows:

Pressure impulse method S cycle	134 °C (273.2 °F)	Minimum sterilisation time	3.5 - 18 minutes
Vacuum method B cycle	134 °C (273.2 °F)	Minimum sterilisation time	3.5 - 18 minutes

- ▶ Follow the instructions of the autoclave manufacturer regarding the sterilisation of this medical products for autoclaves with S cycle method.

Pursuant to U.S. and Canadian standards:

Pressure impulse	134 °C (273.2 °F)	Minimum sterilisation time	3.5 - 10 minutes
Pre-vacuum	134 °C (273.2 °F)	Minimum sterilisation time	3.5 - 10 minutes
Gravity	134 °C (273.2 °F)	Minimum sterilisation time	15 minutes

## 4 Installation and commissioning

### **WARNING! Danger from non-sterile products!**

Risk of infections for treating personnel and patients.

▷ Before first commissioning and after each use the medical product must be processed appropriately and if necessary sterilised.

### **NOTE! Danger during commissioning of the turbine after storage in very cold place!**

Possible malfunction of the turbine.

▷ Bring massively cooled products to a temperature between 20-25 °C (68-77 °F) before commissioning.

### **NOTE! Damages by polluted and moist cooling air!**

Polluted and moist cooling air can lead to malfunction and premature wear of the bearings.

▷ Provide dry, clean and uncontaminated cooling air pursuant to ISO 7494-2.

### **NOTE! Missing or damaged gaskets!**

Missing or damaged gaskets lead to premature wear or malfunction. All 5 gaskets must be applied and must not be damaged.

▷ Check turbine coupling.

## 4 Installation and commissioning

To operate the turbine an operating pressure of at least 2.1 bar (30 psi) is necessary!

▷ Check air pressure.

### Pressure values for turbine

Driving air	2.1 - 4.2 bar (30 - 60 psi)
Recommendation	2.8 bar (40 psi)
Return air	< 0.5 bar (< 7 psi)
Water	0.8 - 2.5 bar (11 - 36 psi)
Spray air	1.0 - 4.0 bar (14 - 57 psi)

The air consumption is 39-55 SLPM.

### Supply of coolant

**CAUTION! Heating of the tooth by insufficient amount of water!**

Thermal damage of the dental pulp.

▷ Set amount of water for spray cooling to at least 50 ml/min.

## 4 Installation and commissioning

### Use of diamond and hard metal drill

#### **WARNING! Risk by using wrong drills!**

The use of wrong drills may lead to damages of the tooth.

- ▷ Use drills as intended.
- ▷ Follow manufacturer's instructions for use.
- ▷ Use only drills made of steel and tungsten carbide, whose shafts comply with standard ISO 1797-1, type 3 and fulfil the following criteria:
  - Diameter 1.59 - 1.60 mm,
  - Total length up to 25 mm,
  - Shaft clamping length at least 11 mm,
  - Cutting diameter up to a maximum of 2 mm.
- ▷ Never use drills with worn shafts. They could fall out during treatment and cause injuries.

## 5 Operation

### Connection to units

**WARNING! Risk of burns by hot turbine!**

Burns in the region of the mouth may result from overheating of the turbine.

▷ Never touch soft tissue with the turbine.

**NOTE! Contaminated and moist pressurised air!**

Contaminated and moist pressurised air results in a premature wear of the bearings.

▷ Ensure supply with clean, contamination-free pressurised air pursuant to DIN EN ISO 7494-2.

▷ Screw coupling on turbine hose and tighten.

### Attaching the turbine

**NOTE! An incorrectly screwed on turbine can loosen from the supply hose and fall down!**

Damage to the turbine.

▷ Check before each treatment, if the turbine is securely attached to the supply hose by pulling carefully.

## 5 Operation

### **NOTE! Activation of the foot switch while attaching the turbine!**

Damage to the turbine and supply hose.

▷ Do not attach turbine with activated foot switch.

### **NOTE! Turbine not exactly attached on the coupling!**

Inexact coupling can destroy the high-pressure lamp of the coupling device or reduce its life span especially when switch off is delayed.

▷ Check for secure placement in the coupling by pulling on the turbine.

1. Attach the turbine straight on the coupling in axial direction (e.g. CONNEX) and push until it audibly locks into place.
2. Check for secure placement in the coupling by pulling on the turbine.

The turbine is also compatible with: KaVo MULTiflex® couplings.

## 5 Operation

### Pulling off the turbine

**NOTE! Risk by activation of the foot switch while pulling off the turbine!**

Damage to the turbine and supply hose.

▷ Do not pull off the turbine with the foot switch activated.

▶ Hold the coupling and pull off the turbine with a slight turn in axial direction.

### Attaching the drill

**WARNING! Injury by sharp edges!**

Infections or cuts.

▷ When testing, attaching and removing wear gloves or other types of finger protection.

1. Firmly press the push-button down with the thumb and at the same time attach the drill as far as possible into the clamping mechanism.
2. Check for firm position of the drill by pulling.
3. Check the lid for firm position with the finger.

## 5 Operation

### Removing the drill

**WARNING! Activation of push-button during operation!**

Damage to clamping mechanism and injury by rotating drill.

▷ Only press the push-button, if the drill no longer rotates.

**WARNING! Heated turbine head by damaged turbine!**

Burns of soft tissue.

▷ Avoid any contact of the turbine head with soft tissue.

**CAUTION! Place instrument with attached drill into holder!**

Injury and infections by the drill. The turbine may fall down.

▷ Remove the drill from the turbine after each concluded treatment.

1. Ensure that the drill no longer rotates.
2. Firmly press the push-button with the thumb down and at the same time pull the drill out.

## 6 Maintenance and repair

### **CAUTION! Risk of injuries and damage to material by improper maintenance!**

- ▷ Have maintenance work only performed by an authorised dealer.
- ▷ Have the device sent to maintenance once a year after the guarantee period has elapsed or after 250 processing cycles.

To guarantee a flawless function of the medical product:

- ▶ Treat the medical product in accordance with the processing methods described in the instructions of use and use the care products and maintenance system described therein.
- ▶ Determine an internal maintenance interval for the practice, at which the medical product is evaluated by an authorised dealer in terms of cleaning, maintenance and function. This maintenance interval depends on the frequency of use and must be adapted accordingly.

### Care products and maintenance systems

### **CAUTION! Sharp milling cutter or grinder in handpiece!**

Risk of injury by sharp and/or pointy milling cutter or grinder.

- ▷ Remove milling cutter or grinder.

### **NOTE! Premature wear and malfunctions by improper maintenance and care!**

Reduced life span of the product.

- ▷ Regularly perform proper maintenance.

## 6 Maintenance and repair

### Care with maintenance systems: *STATMATIC* / *STATMATIC PLUS*

The medical product must be maintained after each cleaning and before each sterilisation.

1. Remove milling cutter or grinder.
2. Maintain medical product in accordance with instructions for use.

#### Collet care

Perform collet care once a week, see instructions for use "*STATMATIC* / *STATMATIC PLUS*", Chapter "Collet care program".

## 6 Maintenance and repair

### Care with dental care spray

The medical product must be maintained after each cleaning and before each sterilisation.

1. Remove milling cutter or grinder.
2. Cover medical product with maintenance bag.
3. Place medical product on the cannula and activate spray button for one second.

### Collet care

Perform collet care once a week.

- ▶ Remove milling cutter or grinder and spray with the tip of the spray nozzle into the opening.

## 7 Guarantee

We offer a guarantee to the user covering all functional, material and manufacturing defects. This guarantee is valid for a period of 2 years from invoice date (the guarantee period may vary in your country, please see your purchasing documents). In the case of justifiable claims we or an authorised representative shall fulfil the guarantee obligations of the company by repairing or replacing the product free of charge. All other claims of any kind, in particular compensation for damages and interest receivable, are excluded from this guarantee.

We hold no responsibility for damage to materials or persons or their consequences, which are caused by the following:

- excessive wear,
- improper handling,
- non-compliance with the instructions of installation, use and maintenance,
- unusual chemical, electrical or electrolytic influences and
- improper connection to air, water and current supply. The guarantee does not extend to flexible glass fibre connectors or parts made of plastic.

## 7 Guarantee

This guarantee can become null and void, if damages and their consequences are caused by the improper use of the product or modifications, which were made by persons unauthorised by us. Claims from this guarantee are only taken in consideration, when invoice or delivery slip are submitted together with the product, which clearly indicate the purchase date as well as the product designation and serial number.

(de = Original / en = translation)