FIG-NR. / FIG-NO.


PRODUKTINFORMATIONEN / PRODUCTINFORMATIONEN

LUS05

LUSTER® EXTRAORAL TWIST KIT

LUS06

LUSTER® INTRAORAL TWIST KIT

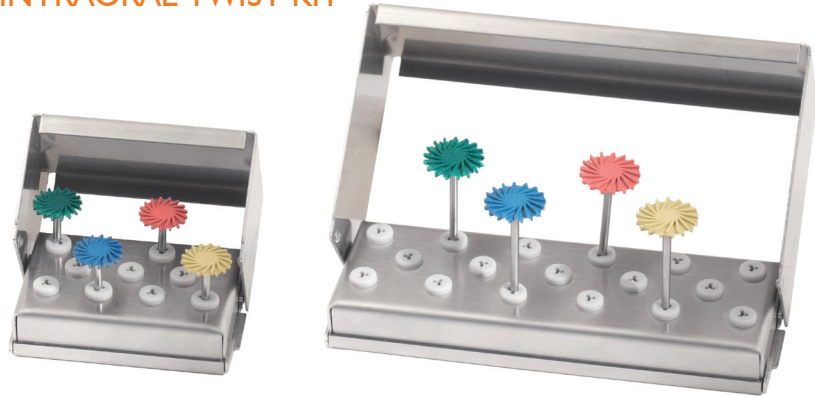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*Luster® Twist Kits have been especially developed for extra- and intraoral high-gloss polishing of all ceramic restorations. The flexible polishing lamellae make it possible to adapt to any surface structure and also render flawless results in areas that are difficult to reach, such as occlusal surfaces. Due to designs that are gentle on materials, the original surface structure of the restoration is maintained. In addition, the grades of diamond grit separated into four parts provide for short processing times and extremely high levels of durability. All instruments of the Luster® Extraoral Twist Kits (LUS05) have a straight handpiece shank and all instruments of the Luster® Intraoral Twist Kits (LUS06) have a right angle shank.*

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*The Twist Polishers adapt with their flexible lamellae to any type of surface structure. With pre-polishing step roughness can be reduced for the final high-shine polishing. You get brilliant results on your ceramic restorations within the shortest time with the usage of four polishers. We recommend using the Twist Polishers at a speed of 7,000 up to 12,000 rpm and with low contact pressure.*